I.FLY

 English 4

 Italiano 8

 Deutsch 12

 Español 16

 Français 20

 Nederlands 24

 Dansk 28

 Polski 32

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 Magyar 40

Wireless Machine for Permanent Make Up, Scalp Micropigmentation, Medical Tattoo, Collagen Bio-Induction® treatments

Warranty

The device has a 12 (twelve) months warranty against any defect referring to production, from the date of purchase. Please keep the invoice as document valid for warranty and refer to the serial number engraved on the device for any communication to the distributor or Clinita headquarter.

For any communication, please always contact your distributor or the local official Clinita partner.

Problems as follows are not covered in warranty:

- Damages due to wrong installation or operation
- Maintenance done incorrectly
- Modifications or changes of parts and components of the dermographier made by unauthorized personnel and without original parts
- Damage due to contact with water, accidental falls, natural disasters
- Any malfunction in the absence of a regular purchase invoice
- Scratches not affecting the correct operation
- Mechanical damage and damage resulting from voltage surges (high and low voltage)

Notes:

- We recommend a 10-minute break from using the device every two hours
- The I-FLY dermographier only works with the FLY needles. Make sure you have assembled them correctly before use
- Do not use the dermographier with wet hands
- Keep the device out of the reach of children and animals
- Keep the spare battery in a cool dry place

After-sales service:

If you need further information, do not hesitate to contact your local official distributor, or write directly to Clinita at clienti@clinita.it

www.clinita.it

Clinita SRL, Via Bertoneria 39,
31059 Zero Branco, TV - ITALY

Mode:	I-FLY
Input:	5V
Output:	4-12V 800MA
Power:	10W
Stroke:	3mm
Material:	Aluminium
Battery:	1000MA
Charging time:	2-3 hours
USB:	5V charger
Working time:	4-5 hours
Pen+battery weight:	91g
Battery weight:	40g
Pen size:	22x144mm
Charging port:	Type-C

Battery protection

The battery is provided with a "short-circuit" protection. When the battery is in a short-circuit situation, it instantly powers off. Please use the charger to power on and reactivate the battery.

Cleaning surfaces

In the case of a contamination of the surfaces: disassemble the grip and go with the procedure of sterilization. The shaft and the long protective sleeve can be cleaned with a disposable cloth moistened with a cleaning agent or a disinfectant. Clean inaccessible surfaces, such as the thread of the grip can be done with a soft-bristled nylon brush.

O-ring maintenance

In case the O-ring is dry and not soft, please use a drop of lubricant. In case the O-ring is broken, replace it.

Note

Please note to keep away from high temperatures and humidity.

Screen

Rotate to disconnect the battery.

STEP 1

Assemble the needle on the pen.

STEP 2

Press for two second for starting.

STEP 3

To adjust the needle depth, rotate the grip.

This section can be disassembled.

Perform cycles of 121°C for sterilization

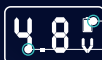
How to sterilize the grip in autoclave.

In case of cross-contamination of the grip proceed with autoclave with a cycle of 20 minutes at a temperature of 121°C and a pressure of 2 bar. After cleaning, make sure the grip is dried before re-using.

Power battery

Battery Charge

Voltage

Screen

Switch: stop and start

Press for 2 seconds for starting up, the default startup power is 7v.

Stop&start: click it once

Pause state:

When the voltage is in stand-by, the voltage is displayed as 0.0 and returns to normal when starting

Up voltage

Down voltage

Adjust voltage for 0.5v

When the battery is in low status, please charge it.

Charging voltage: DC5V+0.2

Charging time:

1.5 to 3 hours

After discharge test:

- 9V can support 1.6 hours of work
- 5V can support 5 hours of work

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